FORLIFE
BERLIN

Stomocur® CE

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Instructions for use EN

Please read the instructions for use carefully, as they contain important information on how to use STOMOCUR® products.

One-piece ileostomy system EN

Intended purpose: Ileostomy pouches are designed for people with an artificial intestinal outlet. Ileostomy pouches are used for the collection of primarily fluid effusions and are designed to be single-use. The contents of the pouch can be emptied via a drainable outlet.

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Preparation

Wash your hands carefully before and after each use.

If the hole opening of the system is too large, the skin is not covered sufficiently. Skin irritation and inflammation can occur owing to long periods of contact with stomal excretions. You should, therefore, determine the diameter of your stoma before use (Fig. 0). Our products are also available in pre-cut hole sizes.

Note: Use **STOMOCUR® Fixing Strips** for extra adhesive surface.

You can use **STOMOCUR® Skin Protection Paste** or **Skin Protection Rings** to smooth out unevenness and avoid skin problems.

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Preparation

Wash your hands carefully before and after each use.

Cut the opening of the skin protection plate to fit the size of your stoma using a pair of scissors (Fig. 1).

Make sure that you pay attention to the size of the skin protection plate cut-to-fit size of the plastic elements (Fig. 2).

⚠ Do not remove the cutting guide film before cutting.

You can use **STOMOCUR® Skin Protection Paste** or **Skin Protection Rings** to smooth out unevenness and avoid skin problems.

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Preparation

Wash your hands carefully before and after each use.

Close the drainable outlet of the ileostomy pouch before attaching it.

To do this, fold the drainable outlet towards the hook and loop fasteners with the help of the plastic elements (Fig. 2).

⚠ Do not remove the cutting guide film before cutting.

You can use **STOMOCUR® Skin Protection Paste** or **Skin Protection Powder** to absorb moisture.

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Attaching the stoma system

In order to support the adhesive properties of the skin protection plate, you should warm it to body temperature.

Remove the cutting aid of the skin protection plate (Fig. 7).

Do not use any products which have a greasing effect when doing this, because they can reduce the adhesive properties of the skin protection plate.

Note: If the skin is moist or wet, you can use **STOMOCUR® Skin Protection Powder**. It absorbs moisture.

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Emptying and cleaning

You can empty the pouch in a hygienic and safe way by using the contents down to a toilet.

Retrieve the closed drainable outlet from behind the non-woven pouch. **Make sure** that you do not inadvertently hold the drainable outlet when retrieving the drainable outlet. This can cause the fastener to open. Open the fastener, unfold the drainable outlet carefully and press the plastic elements together. The contents of the pouch flow out of the drainable outlet into a toilet.

Then move your fingers over the back of the adhesive surface and press this carefully around your stoma.

To ensure adhesion, avoid touching the adhesive surface.

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Removing the stoma system

Before the pouch is removed, make sure you have emptied the contents of the pouch.

Remove the closed drainable outlet from behind the non-woven pouch. **Make sure** that you do not inadvertently hold the drainable outlet when retrieving the drainable outlet. This can cause the fastener to open. Open the fastener, unfold the drainable outlet carefully and press the plastic elements together. The contents of the pouch flow out of the drainable outlet into a toilet.

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